




Striated Bagolini Lenses

Part Number: 346700



Instructions:

1. Adjust Striated Bagolini lenses to match pattern 'A' above if you are using the trial lens mounted Bagolini's. Then have the patient look at a point light source, such as a penlight with both eyes open.
2. Show the patient the reverse side of this card and ask the patient to point to the pattern that best describes what they see.
3. Use the chart below to determine prognosis.
4. If you are using trial lens mounted Bagolini's, you can confirm the diagnosis by adjusting lenses to match Pattern 'B' and repeat.

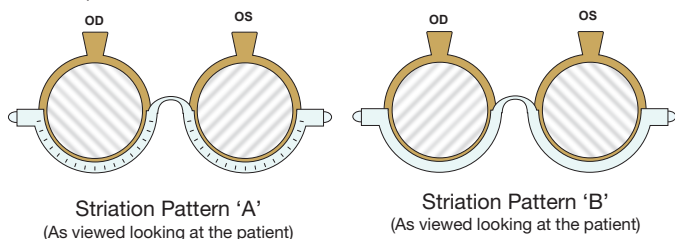

Patient points to Figure:	Patient is wearing Striation Pattern A	Patient is wearing Striation Pattern B
1	Fusion	Fusion
2	Suppression OS	Suppression OD
3	Suppression OD	Suppression OS
4	Fusion w/ Central suppression OS	Fusion w/ Central suppression OD
5	Fusion w/ Central suppression OD	Fusion w/ Central suppression OS
6	Uncrossed diplopia	Crossed diplopia
7	Crossed diplopia	Uncrossed diplopia

Bagolini Key

Figure 1 	Figure 2
Figure 3 	Figure 4
Figure 5 	Figure 6
Figure 7 	